



# Microdermabrasion with Supersonic

SDSITMP101

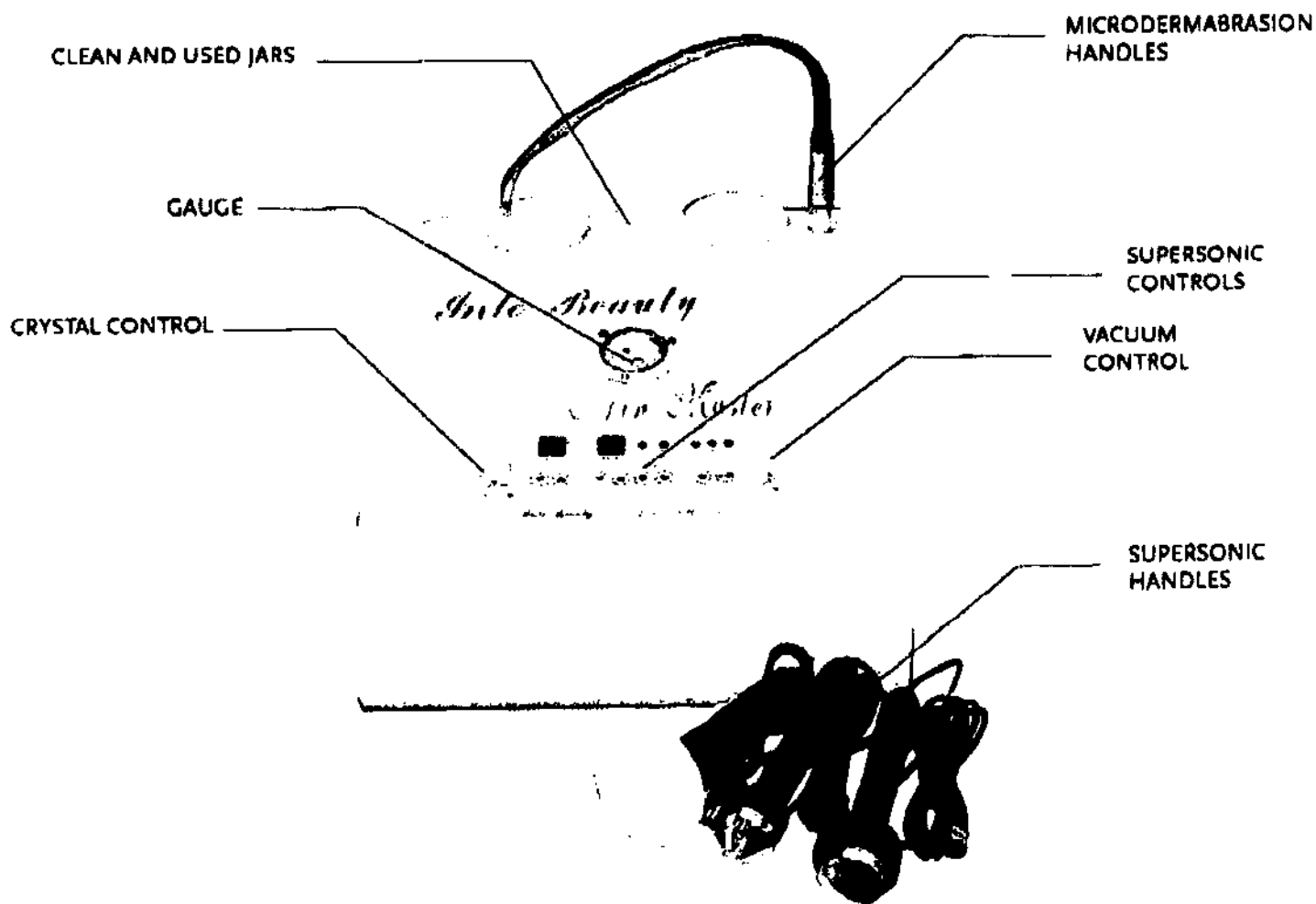


# MAINTENANCE AND SET UP

From the back of the machine you can see the jars for the used and new crystals.

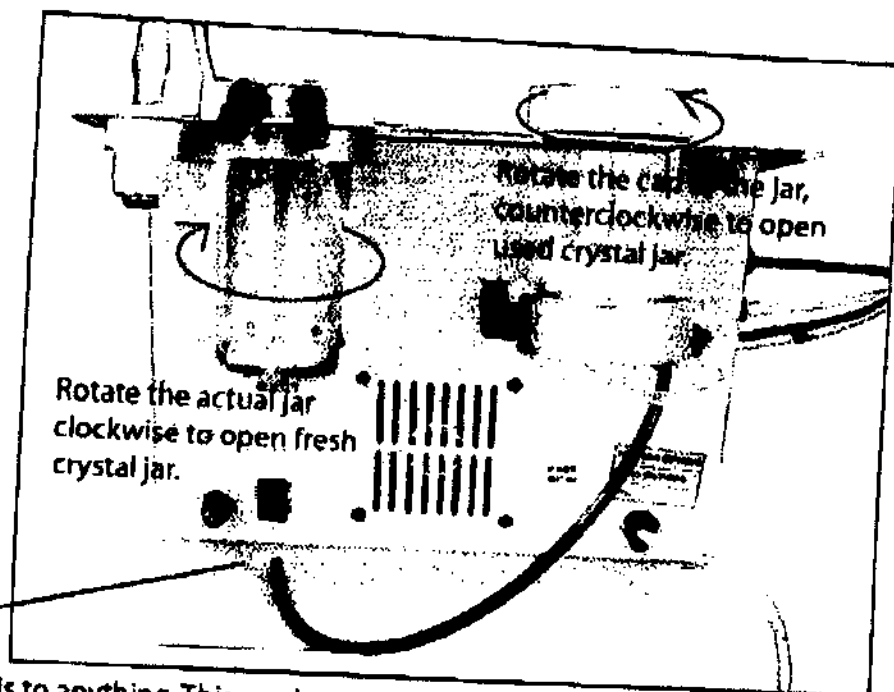


Do not plug this to anything. This can be used to blow away powder crystals.

TO INSERT THE TUBE PUSH THE TUBE TOWARDS THE HOLE THE BLUE RING WOULD AUTOMATICALLY LOCK THE TUBE

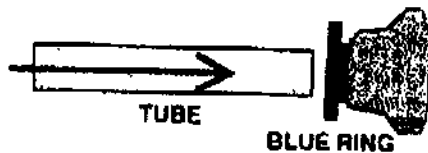

TO REMOVE THE TUBE, PUSH BLUE RING AWAY FROM THE TUBE AND PULL THE TUBE OUT OF THE HOLE.

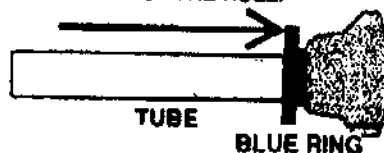

**SET-UP**

1. After the jars are set up, the next step is to attached the handle tubes to the holes on the front panel that says "VACUUM" and "CRYSTALS". Make sure that the tubes are connected securely. The blue tube is for crystals and the white or clear tube is for vacuum.
2. Attached the power cord to the power supply. Make sure the plug the device to a 120 V supply, unless otherwise specified.
3. Turn the Power switch on.
4. Turn the Microdermabrasion swith on as well as the cooling switch.
5. Adjust the timer to the specified time of treatment.
6. At this point you can also adjust the control for the "Crystal" and "Vacuum". This would determine the level of vacuuming and the amount of crystals needed for the treatment.
7. Grab the handle and start the treatment.

**TREATMENT PROCEDURE**

1. Position your client in a comfortable, relaxed posture.
2. Clean area to be treated. Wash with facial cleanser to remove oil, makeup, dirt and grime. To perfect the cleansing process, apply a facial toner onto cotton and gently remove excess residue from face and neck areas.
3. Offer deep exfoliation procedure using a Skin Scrubber as this step is optional.
4. Apply a few drops of Oxiana Serum and massage onto skin.
5. Use the Infusion function (if you have one, other wise apply the product using your hands, massaging the client gently) to allow the serum to penetrate deeper through the natural interstices between the epidermal skin cells (intercellular gap or "Biomatrix") into the deeper layers of the skin. In the "Biomatrix", the active ingredients in the serum are able to develop their positive effects and action most effectively.
6. Begin with Microdermabrasion. Gently move the tip in a sweeping action across the forehead towards the temple area, covering the entire facial area. When treating the neck area, run the tip gently and quickly from under the chin to the collarbone using overlapping strokes.
7. At completion of Microdermabrasion, wipe away any crystal residues that remain on the skin with a tissue and gently cleanse the area with cotton pads, immersed in cold water. (\*As an option, you can cancel the crystal mode and use suction only)
8. Apply a facial toner onto cotton and gently remove excess residue from face and neck areas.
9. Using a damp cotton wool, apply a few drops of the Revitamine® RHR Complex Serum while administering a gentle relaxing massage. Continue with massage for the next 5 minutes.
11. Mix together the Deep Invigorating Tablet Masque in Dermafix Activator Solution with the Oxiana Serum.

(\*Other antiaging / active principles serums can be substituted in order to accelerate the healing process and to allow the easy penetration of the ingredients into the skin).

13. Apply RHR Serum to facial skin until deep penetration is complete in order to hasten the skin's healing process.
14. Apply to Solar/C SPF 30 to face and neck area for complete sun protection

**TIPS**

- Sanitize and insert a new disposable tip on the Microdermabrasion head.
- Press the Dermabrasion Switch and test the level of suction and crystal flow . Test on your hand first to make sure that this is at a comfortable level for your client. If not, it is easy to adjust.
- These requires a maintenance every 3 months depending on the usage of this machine. Please ask your sales representative about extended warranty.

**X. Ultrasonic****Ultrasonic****Operation Method:**

1. Make sure the power cord is plug.
2. Turn the power switch on. On some machines the power switch is at the back and you may need to switch to "II".
3. Attached all three probes or handles. "I" is for the eye and lip area. "II" is for the face. "III" is for the body.
4. Select work time by pressing up or down button.
5. Depending on the skin type, choose a combination of intensity and mode. Intensity can range from 0-9 and can be adjusted using the up or down button. Mode is either "Continuous" or "Impulse". We recommend to use Continuous Mode with Intensity 7 for Healthy Skin. Use Impulse Mode with Intensity 9 for problem skin.
6. Click the small button on the handle or probe to start the treatment.
7. Turn off the power when treatment is done.

**Caution:**

1. Use 110 V unless otherwise specified.
2. If there is any feeling of discomfort due to the heat coming from the probes, decrease the intensity or turn off the machine momentarily.
3. Impulse wave is recommended for problem skin.
4. Do not let the probes fall while in treatment.
5. Always pull the power cord when treatment is done.